## Dear Valued Partners:

We are glad to announce that INTEC product of Syphilis (TP) ELISA kit and Syphilis (TP) Rapid kit and Syphilis(TP) Trust kit have been evaluated by the China CDC and achieved exceptional results. Please find below a quality report from the National Center for STD (Sexually Transmitted Diseases) Control, China CDC.

Official website link as listed below for your reference.

http://www.ncstdc.org/show.asp?id=1651(Test report in Chinese)

Syphilis (TP) ELISA kit evaluation results:

Table 1 Specificity antibodies reagent evaluation results of TP

Method	MFR.	Sample Size	Sensitivity (%)	Specificity (%)	Overall Score(%)	Карра	CV%
ELISA	InTec	307	98.31%	100.00%	99.35%	98.62%	9.52%
	Kehua	307	97.46%	100.00%	99.02%	97.93%	6.05%
	Livzon	307	97.46%	100.00%	99.02%	97.93%	10.49%
	Kinghawk	307	97.46%	100.00%	99.02%	97.93%	11.95%
	Wantai	307	96.61%	100.00%	98.70%	97.23%	4.20%
	BGI∙GBI	307	96.61%	100.00%	98.70%	97.23%	5.25%
	Sunway	307	96.61%	100.00%	98.70%	97.23%	11.30%
	Kewei	307	95.76%	100.00%	98.37%	96.53%	6.13%

Syphilis (TP) Rapid kit evaluation results:

Table 2 Specificity antibodies reagent evaluation results of TP

Method	MFR.	Sample Size	Sensitivity (%)	Specificity (%)	Overall Score (%)	Карра	CV%
Rapid	Biotest	307	96.61%	100.00%	98.70%	97.23%	/
	InTec	307	96.61%	100.00%	98.70%	97.23%	/
	Wantai	307	96.61%	100.00%	98.70%	97.23%	/
	Alere Medical	307	96.61%	99.47%	98.37%	96.54%	/
	Abon	307	94.92%	100.00%	98.05%	95.83%	/
	Wondfo	307	94.92%	100.00%	98.05%	95.83%	/

Fax: +86 592 6519161 http://www. intecasi. com

Tel: +86 592 6807188



Syphilis (TP) Trust kit evaluation results:

Table 3 Non-treponemal antigen serological test evaluation results

Method	MFR.	Sample Size	Sensitivity (%)	Specificity (%)	Overall Score (%)	Карра
RPR	Kehua	310	99.00%	97.14%	97.74%	94.90%
TRUST	InTec	310	98.00%	99.52%	99.03%	97.78%
	Rongsheng	310	97.00%	98.57%	98.06%	95.57%
	Kinghawk	310	96.00%	99.52%	98.39%	96.28%
	Wantai	310	96.00%	99.52%	98.39%	96.28%

In 2017, InTec delivered over 500 million tests to healthcare professionals in more than 80 countries, and we strive for continual improvement through partnerships with health authorities and experts in the management of infectious diseases. It is our vision that through close cooperation with these global stakeholders, InTec will continue to be a key contributor to reaching the goals of hepatitis elimination, HIV 90-90-90, and other ambitious targets set by the foremost authorities in public health.

Tel: +86 592 6807188

Fax: +86 592 6519161

http://www. intecasi. com

Global Sales and Marketing Department InTec PRODUCTS, INC. (XIAMEN)